



## Congratulations on purchasing a MediCap USB200!

Please see the “Quick Start” section in the User Guide. Also, please note:

### ► If there is no video on your monitor...

Please ensure that the video system is set to match your medical device (**NTSC** or **PAL**). Press the “Menu” button. Select “Setup” then “Input Options.” **NTSC** is used in the U.S., Canada, Mexico, South America, Japan, Taiwan, and Korea. **PAL** is used in Europe, China, India, Africa, and the Middle East.

### ► If the videos will not play on your computer...

Please install the “Media Player Upgrade” from the supplied CD. You can also download it from [www.medicapture.com](http://www.medicapture.com). This software allows your media player to play MPEG-2 video. The USB200 records video in this format which is the industry’s highest quality format and is preferred for medical applications.

### ► Use MediCapture brand USB flash drives...

MediCapture flash drives are guaranteed to be compatible with your USB200. Drives from other manufacturers may not be compatible with your device and may not meet the high performance standards required for recording images and video. MediCapture USB flash drives are available at [www.medicapture.com](http://www.medicapture.com).