



Congratulations on purchasing a MediCap USB300!

Please see the “Quick Start” section in the User Guide. Also, please note:

- ▶ **WARNING! Do not select “Record to Internal Hard Drive” unless you have connected your USB300 to a network. The internal hard drive is for networked systems only.**

Please keep the default setting (“Record to External USB Drive”) unless you have first setup a network connection. The internal hard drive can only be accessed through a network connection; the network is the only method for moving videos off the internal hard drive for archiving. Non-networked systems can use the default method of recording to the supplied USB flash drive or optional USB hard drive.

- ▶ **Front panel monitor is for navigation, not diagnosis...**

You should always connect your USB300 to an external monitor. Note:

- At some resolutions, the front panel image may look “squeezed.”
- At 1920 x 1080 resolution, the front panel image is color bars rather than menus or live video.

The image will be displayed normally on your external monitor.

- ▶ **If HD videos do not play on your computer...**

To enable HD video playback on your computer, you can install the free “Media Player Upgrade” from the supplied CD. Or you can download it from www.medicapture.com.