

## How to Upgrade a MVR Lite 4K

### Step 1: Check the actual firmware version.

- Switch-on MVR Lite 4K.
- After the boot sequence you can access the info page by using the remote control.
- Press the outer info page buttons.
- Now you can see the actual firmware version, e.g. SYS 160307
- If the latest MediCapture Firmware Update Report recommends an update of this firmware version, please switch-off MVR Lite 4K and continue with step 2.

### MVR Lite 4K Remote Control



Page Up/Down Button:  
Accesses and Closes the  
MVR Lite 4K Info Page

### Step 2: Upgrade the firmware.

- Create an „Upgrade Flash Drive“. To do so, delete all files and make sure the USB flash drive is formatted in FAT32.
- Please access the link provided to you by your sales representative to download the Upgrade File.
- Download the latest firmware and copy it to the USB Flash Drive. The update file is called iMave-xxxxxx.upd.
- Make sure that a monitor is connected to the HDMI output of the MVR Lite 4K so you can see the displayed information.
- Insert the USB flash drive into the front panel.
- Press and hold the "Video Button" (in the middle) on the front and switch-on MVR Lite 4K. After 10 seconds you can release the front button.
- Observe the monitor display until MVR Lite 4K has fully booted and you can see the icons.
- Check correct firmware version – see Step 1. Check date and time settings also.
- Switch-off the MVR Lite 4K. It is ready-to-use now.