

MediCap[®] USB300 Printing Guide

Applies to Firmware 110701 and Higher¹

The MediCap USB300 can print images on the Sony UP-DR80MD medical color printer². For detailed information on this printer, refer to documentation available from Sony.

How to Connect the Printer

Connect the printer to one of the USB ports on the back of the USB300 using the cable included with the printer. Plug the printer's power cable into a wall outlet. Ensure that the printer is turned on and is loaded with ink and paper. No configuration is needed on the printer or the USB300.

How to Print

When printing, you will load images into a "print queue" and then print a page containing the images. While following these steps, you will only be pressing buttons on the USB300; no buttons are pressed on the printer.

1. After recording some images, enter "review mode" by pressing the *Select* button.
2. Use the forward and back buttons (arrow buttons) to display an image you want to print.
3. Press the *Menu* button. The Print screen will be displayed. This screen contains three choices: Back, Print, and Quick Print.
4. Select Print. This will load the first image into the print queue.
5. Select Back. This will return you to review mode.
6. Repeat steps 2 through 5 to load more images into the print queue.³
7. When the last image is loaded into the print queue, the page containing the images is automatically printed and the print queue is cleared.

Notes on Printing

- To print images before you fill up the print queue, select Quick Print after you load the desired image (or images). This is useful if you want to print just one image on a page.
- The case number is printed at the top of each page and the image number is printed next to each image. If you are using the patient information feature, patient data is printed at the top of each page (last name, first name, etc.).
- The lower-left corner of the Print screen displays the number of pages remaining on the roll of paper in the printer.
- Normally, images are printed from the current patient session. To print images from previous patient sessions, use the Patient Review feature.⁴ To use this feature, go to the Advanced Menu and select "Storage Management" and then select "Patient Review." This will allow you to view images from previous patient sessions. When you see an image you want to print, press the *Menu* button (step 3 above). For more information on the Patient Review feature, see the *USB300 User Guide* which is available in the Support section at medicapture.com.
- If the printer is not plugged into the USB300, the Print screen will not be displayed when you press the *Menu* button while in review mode (step 3 above).

¹ This document applies to USB300 video recorders running firmware version 110701 and higher. To check your firmware version, use the Current Settings option in the Advanced menu. For documentation on earlier versions of the firmware, see medicapture.com/support. For help with upgrading the firmware, please send an email to support@medicapture.com.

² Other printer models are not supported at this time.

³ If you are using A4 paper, the print queue holds 3 images. If you are using 8.5" x 11" paper, the print queue holds 2 images.

⁴ Not available on units that are configured to record onto a network drive.